# Medical catalogues

## D - DRUGS / MEDICAMENTS 3

<b>DEXT External use - Usage externe .....</b>	<b>3</b>
<b>DINJ Injectable drugs - Médicaments injectables.....</b>	<b>4</b>
<b>DORA Oral drugs - Médicaments oraux .....</b>	<b>5</b>
<b>DVAC Vaccines - Vaccins .....</b>	<b>7</b>
<b>EDDC Drug dispensing &amp; compounding equipment - Equipement pour l'administration &amp; préparation extemporanée des médicaments .....</b>	<b>Error! Bookmark not defined.</b>

## S - RENEWABLE MEDICAL SUPPLIES / MATERIEL MEDICAL RENOUVELABLE 10

<b>SCTD Catheters, tubes &amp; drains - Sondes et drains.....</b>	<b>10</b>
<b>SDRE Dressings - Pansements .....</b>	<b>12</b>
<b>SINS Injection material - Matériel d'injection .....</b>	<b>12</b>
<b>SMSU Small medical supplies - Petit matériel médical.....</b>	<b>15</b>
<b>SPAP PAraPharmacy - PAraPharmacie .....</b>	<b>16</b>
<b>SSCO Surgical consumables - Consommables chirurgicaux.....</b>	<b>16</b>

## E - Medical equipment - Equipement médical 17

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#### In WHO Essential Medicines List

Misoprostol 25µg has been included in MSF's obstetric guidelines for several years as an alternative to Misoprostol 200µg for the induction of labour in patients with term pregnancies. The oral formulation (DORAMISP25T) was previously available as a standard item, at a dose of 25µg every 2 hours.

However, due to cost and mainly availability issues, the oral tablet may no longer be available in MSF in the coming years. It is replaced by the Misoprostol 25µg vaginal tablet, administered according to the WHO protocol with 25µg every 6 hours until strong contractions occur.

The same contraindications that apply to the oral Misoprostol 25µg also apply to the vaginal tablet (main contraindications being a history of previous caesarean section(s)).

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In WHO Essential Medicines List

The new combi-pack includes:

- 1 powder vial containing 60 mg of artesunate
- 1 ampoule with 3 ml of 0.84% (8.4mg/ml) sodium bicarbonate ( $\text{NaHCO}_3$ ) + 2% (20mg/ml) arginine for reconstitution

This new injectable artesunate formulation eliminates the need for separate reconstitution and dilution of the powder. It simplifies and speeds up administration, reducing the risk of preparation errors.

The 3 vials presentation (DINJARTS6V-) and the new 2 vials presentation (DINJARTS6V3) use the same weight-based dosing.

Posters with instructions for reconstitution and administration are available in English and French.

Missions should gradually shift from DINJARTS6V- to DINJARTS6V3. To ensure a smooth transition between the two artesunate formulations, estimate the forecast and assess the manufacturer's production capacity, this product is reserved for Nigeria, CAR and DRC missions for one year.

Having both formulations simultaneously at field level is discouraged.

Justification: restricted to Nigeria, CAR and DRC missions for one year. Other missions should contact their OC malaria referent for approval before ordering.

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In WHO Essential Medicines List

Three new formulations (dispersible tablets) have been introduced to replace the normal tablets, along with two new concentrations.

A comparison of the commercial conditions between the two sources (D-ARTEPP® and Eurartesim®) shows that D-ARTEPP® offers better conditions, including the option to order smaller quantities.

The posology and quantity of blister, based on weight, have been calculated in line with WHO recommendations, as follow:

Code	Strength	Presentation	Weight	Dose recommended by WHO	Qty of blister
DORADHAP1TD1	20 / 160 mg	3 disp. tabs / blister	5-8 kg	1 tab / day for 3 days	1
			11-17kg	2 tabs / day for 3 days	2
DORADHAP2T1	30 / 240 mg	3 disp. tabs / blister	8-11 kg	1 tab / day for 3 days	1
			17-25 kg	2 tabs / day for 3 days	2
DORADHAP3TD1	40 / 320 mg	3 disp. tabs / blister	11-17 kg	1 tab / day for 3 days	1
			25-36 kg	2 tabs / day for 3 days	2
DORADHAP6T1	60 / 480 mg	6 tabs / blister	17-25 kg	1 tab / day for 3 days	1/2
			36-60 kg	2 tabs / day for 3 days	1
DORADHAP7T1	80 / 640 mg	6 tabs / blister	25-36 kg	1 tab / day for 3 days	1/2
			60-80 kg	2 tab / day for 3 days	1
			>80 kg	2.5 tab / day for 3 days	2

Presentations must be ordered for all weight categories.

The quantity to be ordered should be calculated on the number of blisters (1 blister = one treatment), not on the number of tablets.

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Not included in WHO Essential Medicines List

In line with the new Paediatric Care and NCD guidelines, Montelukast 5 mg is introduced as STD for add-on maintenance therapy of chronic asthma in children aged 5 to 12 years who do not respond to inhaled corticosteroids (CSI, e.g. inhaled beclometasone) and Long-Acting Beta<sub>2</sub> Agonist (LABA, e.g. inhaled salmeterol). In that case, Montelukast will be added to the ICS + LABA regimen.

Justification: indicated as an adjunct to medium-dose ICS and LABA if there is a poor response to ICS+LABA alone.

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All those presentations become NST outdated and are replaced with new formulations and concentrations.

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Due to supply issue, the normal tablet becomes NST, and it is replaced by the vaginal tablet (DEXTMISP25TV), available as STD.

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Not included in WHO Essential Medicines List

New vaccine prequalified by WHO, indicated for the active immunisation of children aged 5 to 36 months for malaria prevention due to *Plasmodium falciparum*.

Recombinant, adjuvanted vaccine composed of the R21 malaria antigen combined with Matrix-M1 (adjuvant). The vaccine has now been deployed to many countries where MSF operates, and MSF will be participating in implementation efforts and/or operation research.

Malaria vaccines are considered as “an additional tool” in the malaria prevention strategy and should not replace existing prevention measures.

Since the malaria vaccine is a new vaccine, pharmaco-vigilance will be an important focus in countries introducing the vaccine. Check the country's malaria vaccine implementation plan to determine if and how MSF can provide support.

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In WHO Essential Medicines List

Equine serum globulins indicated for the treatment of African snakebites envenomations.

- EchiTab-ICP (DVACIMAS19V) is indicated in all countries (except Ethiopia and South Sudan) in case of haematotoxin or cytotoxic symptoms. Effective at neutralizing venoms from the most important viperids *Echis* spp. (hemotoxic) and *Bitis arietans* (strong cytotoxic), and partly effective on some cytotoxic elapids (*Naja* spp. cobra).
- SAIMR (DVACIMAS3A-) is reserved for use in Ethiopia and South Sudan, in case of neurotoxic or critical cytotoxic syndromes. Best option for neutralizing neurotoxic cobras (*Naja* spp.) and mambas (*Dendroaspis* spp.), and rescue option for cytotoxic vipers (*Bitis* spp.).

Follow the clinical and decisions algorithms published by the NTD platform in 2024:

- Clinical algorithm snakebite antivenoms most African countries (except\_ETH\_SSD)

- Clinical algorithm snakebite antivenoms - ONLY Ethiopia & South Sudan

Justification: to be used under medical supervision. Monitor their use and implement a reporting system (bites, type of species, countries).

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The vial and its diluent become NST and discontinued due to discontinuation of production by the manufacturer. Depending on the country's preferences, it will be replaced by:

- (DVACVMENA2VD) VACCINE MENINGOCOCCAL A CONJ. 3-24months, 1dose, multid. Vial, or
- (DVACVMENX1VD) VACCINE MENINGITIS ACYWX ps. CONJ., 1 dose, multidose, vial. *MenFive 5µg®*

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Becomes NST and discontinued due to discontinuation of production. Replaced by either:

- (DVACVRAB2S-) VACCINE RABIES, CCV, cell cult., monod., vial + syr. diluent, or
- (DVACVRAB3V-) VACCINE RABIES, CCV, cell culture, monodose, vial

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Becomes NST and discontinued as no longer available for MSF procurement. In case of local purchase, it will become NSL.

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**EDDC****Drug dispensing & compounding equipment**

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Due to manufacturing issues, the 500ml square bottles are no longer available and have been replaced with round bottles.

Consequently, the holders for square bottles become NST and have been replaced with holders for round bottles. If there is still stock available in the mission, the holders for square bottles (EDDCHOLD5W1 & EDDCHOLD5B1) are compatible with the 500 ml round bottles, but not vice versa.

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# S - RENEWABLE MEDICAL SUPPLIES / MATERIEL MEDICAL RENOUVELABLE

## SCTD

### Catheters, tubes & drains - Sondes et drains

#### NEW / NOUVEAU

SCTDCANN2I-	STD	NASAL OXYGEN CANNULA, 2 prongs + tube, infant LUNETTES A OXYGENE, 2 embouts + tube, nourrisson	MD Class IIa
-------------	-----	---	--------------

Additional size for the nasal cannula for children 1 month to 2 years old, inadequately sized nasal cannulas which may impact on treatment/oxygen delivery and contribute to other complications such as pressure sores. The other STD sizes are: adult (SCTDCANN2A-), child (SCTDCANN2CL), neonate (SCTDCANN2N-) and premature (SCTDCANN2PL).

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The disposable Guedel cannulas become sterile as they are more available on the market. They replace the non-sterile cannulas, also in the different medical kits and modules (same quantities)

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They are the replacement of: / Elles remplacent:

SCTDAIRGN040	NST	OROPHARYNEAL AIRWAY, s.u., non ster., 40mm, ID 3.0mm, pink CANULE OROPHARYNGEE, u.u., non stér., 40mm, DI 3.0mm, rose	
SCTDAIRGN050	NST	OROPHARYNEAL AIRWAY, s.u., non ster., 50mm, ID 3.5mm, blue CANULE OROPHARYNGEE, u.u., non stér., 50 mm, DI 3,5mm, bleu	
SCTDAIRGN060	NST	OROPHARYNEAL AIRWAY, s.u., non ster., 60mm, ID 4.0mm, black CANULE OROPHARYNGEE, u.u., non stér., 60mm, DI 4.0mm, noir	
SCTDAIRGN070	NST	OROPHARYNEAL AIRWAY, s.u., non ster., 70mm, ID 4.0mm, white CANULE OROPHARYNGEE, u.u., non stér., 70mm, DI 4.0mm, blanc	

SCTDAIRGN080	NST	OROPHARYNGEAL AIRWAY, s.u., non ster., 80mm, ID 4.5mm, green CANULE OROPHARYNGEE, u.u., non stér., 80mm, DI 4,5mm, vert
SCTDAIRGN090	NST	OROPHARYNGEAL AIRWAY, s.u., non ster., 90mm, ID 4.5mm, yellow CANULE OROPHARYNGEE, u.u., non stér., 90mm, DI 4.5mm, jaune
SCTDAIRGN100	NST	OROPHARYNGEAL AIRWAY, s.u., non ster., 100mm, ID 5.0mm, red CANULE OROPHARYNGEE, u.u., non stér., 100mm, DI 5,0mm, rouge

SCTDPDSFS01-	STD	PLEURAL DRAINAGE SYSTEM, 1150ml, ster.s.u. (Sinapi XL1150SC) SYSTÈME DRAINAGE PLEURAL, 1150ml,stér.u.u. (Sinapi XL1150SC)	MD Class IIa
--------------	-----	--	--------------

The proposed new model (Sinapi medical - 1150SC - Cardiac&Thoracic use) is cheaper than the Pleurevac from Teleflex. It also is suitable for adults and children and allows to regulate the degree of suction. It incorporates a site to insert a transfusion set to perform an autologous blood transfusion and has a non-return valve if tilted or above patient.

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A sterile, flexible plastic pouch with an adhesive flange (with a gender-specific shape) intended to be attached to the skin around the genitalia to collect urine from a child. It is not designed with an opening for urine drainage and is typically used for sampling urine in an aseptic way.

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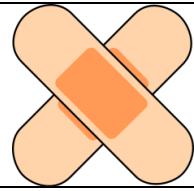
This article rarely used, is not included in any nursing protocol, not part of any kit. It is intended for oxygen administration, but we use mainly the nasal cannula and the oxygen masks. The article remains available for ordering as an NST.

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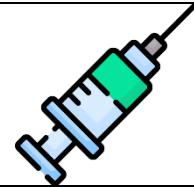
A smaller size of Plaster of Paris bandage, mainly for children. The 10cm is the most common size available. The STD adult size is 15 cm.

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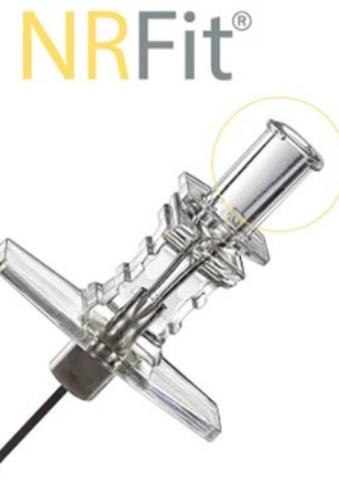
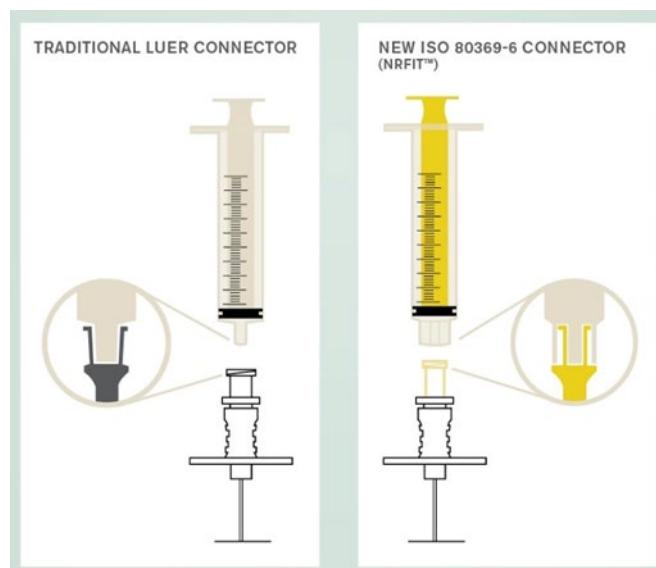
Change of specifications of the safety IV catheter 24G as we have received many quality complaints, from different brands. A technical cause was never identified, we try to facilitate the handling of the IV catheter during the insertion procedure by removing the injection port. The 24G catheter is quite small, so the side of the injection port complicates the handling and manipulation of the catheter during the insertion and fixation.

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Is replacing Remplace	SINSIVCST24W1	NST	SAFETY IV CATHETER, tip, 24G x 19 mm, wings, IP, yellow CATHETER IV SECURISE, tip, 24G x 19 mm, aill., PI, jaune
--------------------------	---------------	-----	---

SINSANPSR1-	<b>STD</b>	RACHI SET NRFit, Whitacre 25Gx90mm, 5ml Lock syringe + acc1 RACHI SET NRFit, Whitacre 25Gx90mm, 5ml Lock seringue +acc1	
SINSNSNRP25I	<b>STD</b>	SPINAL NEEDLE, ANAESTHESIA, NRFit, pencil, 25Gx90 mm, guide AIGUILLE RACHIANESTHESIE, NRFit, crayon, 25G x 90 mm, guide	MD Class III
SINSNSNRP25IL	<b>STD</b>	SPINAL NEEDLE, ANAESTHESIA, NRFit, pencil, 25Gx120mm, guide AIGUILLE RACHIANESTHESIE, NRFit, crayon, 25Gx120 mm, guide	MD Class III
SINSNSNRP27I	<b>STD</b>	SPINAL NEEDLE, ANAESTHESIA, NRFit, pencil, 27Gx90mm, guide AIGUILLE RACHIANESTHESIE, NRFit, crayon, 27Gx90mm, guide	MD Class III
SINSSYNRL03	<b>STD</b>	SYRINGE, s.u., NRFit Lock, 3 ml SERINGUE, u.u., NRFit Lock, 3 ml	MD Class Is
SINSSYNRL05	<b>STD</b>	SYRINGE, s.u., NRFit Lock, 5 ml SERINGUE, u.u., NRFit Lock, 5 ml	MD Class Is
SINSTRNR01-	<b>STD</b>	FILL NEEDLE NRFit, 18G, blunt, 5 µm filter, sterile, s.u. AIGUILLE PUISEUSE NRFit, 18G, mousse, filtre 5µm, stér, u.u.	MD Class Is

ISO-80369 will regulate and standardize small bore connectors used to connect medical devices, components and accessories for the purpose of delivering drugs (and obtaining specimen/sample). Part 6 of this ISO is specific for the neuraxial and major regional anaesthesia systems, the connector for these systems are NRFit.

This new standard "NRFit" will no longer be compatible with the present Luer connector.

While roll-out has been slow the new connection standard will become mandatory. MSF has been conducting pilots with new set incorporating this new connector and collecting feedback from end users.

The proposed NRFit set is custom made according to the MSF protocol for rachi anaesthesia and combines all disposable items in one box. This will replace current practice that is associated with a number of issues from a infection control perspective, and would expedite the procedure. Syringes and other needles will be available as single items in small quantities to avoid opening another set. The supplied rachi needle remains sterile in its primary packaging and can be swapped to a different length/gauge without a loss.

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Arterial line sets are needed for invasive blood pressure monitoring via connection to a transducer which is plugged in to the vital signs monitor. Intra-arterial blood pressure monitoring is indicated in acute care areas where close monitoring or BP is required: use of vasopressors, need for multiple samplings eg for arterial blood gases. Placement is done by trained physicians, order to be justified: PM

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The needle is very long and unlikely to be needed in most of our settings; we prefer to remove it to avoid confusion in ordering. It could still be available as non-standard in case of need (although unlikely). Remains STD: SINSNEAI3104 PEN NEEDLE, 31-32G, 0.23-0.25 x 4-5mm

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Every inpatient (hospitalized) must have proper identification. Used to confirm identity, to minimize risk of errors (for example wrong medication to the wrong patient or swapping babies. SMSUBRAI4WN is tailored to the specific needs for neonates, particularly concerning the right size. Neonates and Tiny babies have unique requirements due to their smaller size and developmental stage, which cannot be met by the existing article SMSUBRAI3W: BRACELET, IDENTIFICATION, write on, plastic, ad/child, white.

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Item will no longer be used for in-patient admission. The blue bracelets remain available and are included in the outpatient nutrition kit KMEDMNUTO31. The red bracelets in the inpatient nutrition kit KMEDMNUTI31 are replaced by the STD white bracelets for inpatients (SMSUBRAI3W-)

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Two new products created in the SPAP family to replace the products previously belonging to the PHYP family as they are used for medical purpose. The former PHYP codes have been archived.

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A new code and label created as the SLASSODC89B5 and DEXOSODC9D5 products were supplied under the same product, regardless their intended use (ophthalmic use and/or lab purposes). The product supplied by ESCs is clearly authorised for ophthalmic use as a medical device, which does not fit into the DEXO qualification scheme. There is a concern regarding the use of a product designated under SLAS coding for human use. The SSCO family already includes other 0.9% NaCl 0.9% presentations for irrigation or wounds cleansing.

SLASSODC89B5 has been replaced by SSCOSODC9A5 in the concerned kits (KMEDMTRA03A, KMEDMSAM1C-, KMEDMLAB15A, KMEDMTRA01A, KMEDMLAB19A).

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# E - Medical equipment - Equipement médical

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New accessories and spare-parts created for the anesthesia ventilator Glostavent Helix EANEANAE5--

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New LED Bulb for the Riester laryngoscope EANELARY7-- LARYNGOSCOPE FO (fibrooptic) + 7 blades

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Creation of different codes for the ultrasound Edge 2 for language specific versions:

- with QWERTY keyboard (English)
- with AZERTY keyboard (French)

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New linear transducer for the M-Turbo ultrasound replacing the EDIMULSA208.

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Article stopped as the transducer is no longer produced by the manufacturer

The alternative is the new transducer **EDIMULSA416**(ultrasound M-Turbo) LINEAR TRANSDUCER L38xi

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Article stopped as the transducer is no longer produced by the manufacturer. No alternative proposed by the manufacturer.

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EDIMDISA3P2	NST	(DI scanner Agfa CR10X) STORAGE PHOSPHOR SCREEN, 24x30cm (scanner IN Agfa CR10X) ECRAN DE STOCKAGE PHOSPHORE, 24x30cm	Thermosensitive: 1525 MD Class IIa
EDIMDISA3P3	NST	(DI scanner Agfa CR10X) STORAGE PHOSPHOR SCREEN, 35x43cm (scanner IN Agfa CR10X) ECRAN DE STOCKAGE PHOSPHORE, 35x43cm	Thermosensitive: 1525 MD Class IIa

The storage phosphor screens become outdated and NST, as AGFA stops producing the Cassettes and the phosphor screens as separate items. Cassettes and screens as a combination remain available as normal under the codes:

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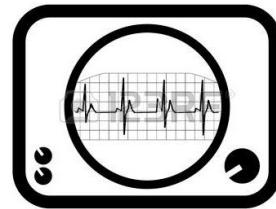
Article stopped as the transducer is no longer produced by the manufacturer. The generic trolley remains available under the code:

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Each equipment is identified with a specific root of 3 letters followed by 1 letter designing if the category of the article

- **+A for the accessories**
- **+C for the related consumables**
- **+S for the different spare-parts**
- **+E for the equipment:** from April 2025 on, the newly created article with the root ending with a **E**, will only include the device itself (equipment only or apparatus) without the additional accessories, consumables....

A category is now existing with the letter P

- **+P for the equipment including the minimal package** of accessories/consumables and spare-parts to have the machine operational

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Portable device allowing to monitor arterial oxygen saturation (% SpO<sub>2</sub>) as well as the pulse rate (beats/minute). This non-invasive continuous measurement is taken thanks to a sensor placed on a finger, a toe or an ear lobe.

The model Masimo RAD-57 replaces the former model Masimo RAD-5 which is no longer manufactured. Most of the accessories and spare-parts are compatible with the new model: check the related articles like accessories, consumables or spare-parts.

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EEMDDRIE700	<b>STD</b>	(drill CD9) HANDPIECE,4505000000 (perceuse CD9) PIECE A MAIN 4505000000	Thermosensitive: TM40 UN3481 Justification: PME MD Class IIa
-------------	------------	--	---

EEMDDRIP7--	<b>STD</b>	SURGICAL DRILL (Cordless Driver 9) +accessories PERCEUSE CHIRURGICALE (Cordless Driver 9) +accessoires	Thermosensitive: TM40 UN3481 Justification: PME MD Class IIa
-------------	------------	---	---

New model of Cordless Driver N°9 to replace the N° 8 which is discontinued. Most of the accessories and spare parts are compatible with the other models. The models are mentioned in the labels of the articles.

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Is replacing Remplace	EEMDDRIE5--	<b>NST</b>	SURGICAL DRILL (Cordless Driver 8) +accessories PERCEUSE CHIRURGICALE (Cordless Driver 8) +accessoires
--------------------------	-------------	------------	---

#### New accessories and spare-parts for the CD9

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New model of the enteral nutrition pump, Flocare Infinity III which is replacing the previous model Flocare Infinity discontinued by the manufacturer Nutricia.

All consumables and accessories are compatible between the old and new model. The enteral nutrition pump enables the nutrient mixture to enter the patient's stomach, by a pipe and a probe. The pump delivers nutrients through pulsation; by using a roller around which is wound a tube.

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Is replacing Remplace	EEMDENPE1--	<b>NST</b>	ENTERAL NUTRITION PUMP (Flocare Infinity) 230V with battery POMPE NUTRITION ENTERALE (Flocare Infinity) 230V +batterie
--------------------------	-------------	------------	---

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For the patient monitors MSF moves from 2 levels of monitors the Dinamap V100 and GE Healthcare B125 to a more layered offer with **3 levels** each corresponding to specific uses

- The vital signs monitor VS8
- The basic vital signs monitor ePM10
- The advanced vital signs monitor ePM12

For a detailed composition of the monitor and supplied accessories, see the online unicat or contact your biomedical referent. The list of accessories, consumables and spare-parts is not definitive, and some new codes are still being created. <https://unicat.msf.org/>

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A device designed for spot check and recording of vital signs and routine observations of one patient: heart rate, oxygen saturation and blood pressure (diastolic and systolic) in a non-invasive manner, with adjustable alarms for these parameters.

The apparatus is used for occasional or periodic readings of vital signs in healthcare settings such as: triage, emergency department, inpatient ward adult and paediatric.

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#### Accessories for the monitor Mindray VS8

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Portable basic Vital signs monitor, used to provide continuous measurement of heart rate, respiratory rate, electrocardiogram, pulse oximetry, temperature and continual non invasive blood pressure.

This type of multiparameter monitor is aimed to be used in : general and advanced emergency departments, specialised inpatient wards such as : PICU/NICUs, basic critical care units, post anaesthesia care units, and during medicalised transport

This monitor replaces certain uses of the monitor B125 of GE Healthcare.

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Advanced Vital signs monitor, used to provide continuous measurement of heart rate, respiratory rate, electrocardiogram, pulse oximetry, temperature, continual non invasive blood pressure, continuous invasive blood pressure, and capnography

This type of monitor is aimed to be used in : operating room, advanced critical care units

This monitor replaces certain uses of the monitor B125 of GE Healthcare.

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## Spare-parts for the ePM monitor series of Mindray

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## NEW Accessories / Consumables / Spare-parts

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New spare-parts created for the oxygen concentrator DeVilbiss 1025KS

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New accessory created for the Cordless Driver 3, 4 and 8 from Stryker

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It is replacing the previous existing tray container

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New accessory created for EEMDESUE10- ELECTROSURGICAL SYSTEM (Lamidey SEAL), 230V + accessories

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It is replacing the bipolar forceps which is no longer available and is now stopped and NST

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New consumable created for EEMDESUE10- ELECTROSURGICAL SYSTEM (Lamidey SEAL), 230V + accessories

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It is replacing the patient split return electrode which is no longer available and is now stopped and NST

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---

EEMDHEPA205	<b>STD</b>	(heating pad Calibed) REMOVABLE COCOON, + cover, reus., 3715 (matelas chauffant Calibed) COCON AMOVIBLE +housse réut 3715
-------------	------------	--

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---

New spare-part created for EEMDVICE4-- INTENSIVE CARE VENTILATOR (Monnal T60), 230V 50-60Hz

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The unidirectional cannula's for the Diamedica CPAP are no longer available. The alternative for it is to use the following Nasal prongs or also called Hudson Cannulas: EEMDCPAC603 / EEMDCPAC604 / EEMDCPAC605 / EEMDCPAC606 / EEMDCPAC607 / EEMDCPAC608 / EEMDCPAC609 / EEMDCPAC610

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---

EEMDDRIE5--	<b>NST</b>	SURGICAL DRILL (Cordless Driver 8) +accessories PERCEUSE CHIRURGICALE (Cordless Driver 8) +accessoires
-------------	------------	---

---

Article is stopped because of the stop of production. It is replaced by Cordless Driver 9 from the same manufacturer, Stryker.

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---

EEMDENPE1--	<b>NST</b>	ENTERAL NUTRITION PUMP (Flocare Infinity) 230V with battery POMPE NUTRITION ENTERALE (Flocare Infinity) 230V +batterie	UN3481 MD Class IIa
-------------	------------	---	------------------------

---

The manufacturer has ended the marketing of this device in March 2024. Corrective maintenance will remain available until 2029. the updated model Flocare Infinity III is proposed as replacement by the manufacturer, all consumables and accessories are compatible.

Replaced by the new model Infinity III **EEMDENPE13-**

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---

Article is stopped as the article is a duplicate of EEMDLOPS325

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---

The Dinamap monitor is stopped and becomes NST as it is no longer produced. It is replaced by the vital signs monitor Mindray VS8 **EEMDMONE22-**

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---

The article is stopped in 2025 because it is no longer produced and it is replaced by the model Masimo RAD-57 **EEMDCAPE2--**

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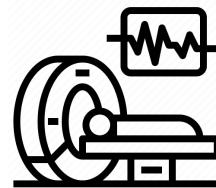
---

Article is outdated and NST as it is no longer produced, no replacement (info supplier: this part cannot be sold separately of the bottle because of the certification. Sealing of the bottle and this part has to be done at manufacturer level)

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**EIMT****Inspection, measuring and test equipment**

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New accessory created for EIMTSIME1-- PATIENT SIMULATOR, Rigel UniSim+PulsR

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**EMEQ****Small medical equipment**

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EMEQNASG1--	<b>STD</b>	NON-PNEUMATIC ANTI-SHOCK GARMENT for PPH care VETEMENT ANTI-CHOC NON PNEUMATIQUE pour p.e.c. PPH	MD Class I
		<p>The Non-pneumatic Anti-Shock Garment (NASG) is a device developed as a temporary measure to gain time, in order to ensure hemodynamic stability and allow patient transfer and/or definitive OH treatment. The NASG is a compression suit made of five neoprene segments that close tightly with Velcro around the legs, pelvis and abdomen. By providing circumferential compression it decreases the vascular volume under the compressed areas, while expanding the central circulation, increasing the preload, peripheral resistance and cardiac output. Furthermore, abdominal, pelvic and uterine vessels' tamponade reduces blood loss. In short, the NASG helps to control bleeding and reverse shock, ensuring circulation to vital organs.</p>	

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Two additional sizes for the tourniquet adapted for children and for babies. They are latex-free and have a release buckle. The adult version EMEQTOUR1-- TOURNIQUET FOR PHLEBOTOMY, elastic, 100 x 1.8 cm remains standard.

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A mains electricity (AC-powered) device designed for total elimination and/or inactivation of microorganisms from medical devices and related products using pressurized steam (i.e., moist heat) as the sterilizing agent; it is used for products non-sensitive to high temperature, water, or steam. It includes a treatment chamber of 85 litres for device placement, and controls; it is intended to sterilize wrapped and/or unwrapped devices.

The 3870 HSG is a hospital grade Class B autoclave, suitable for large medical & dental clinics and operating rooms. Designed to be a fast cycle sterilizer that is mobile, operates 24 hours a day, and can optionally operate independent of building utilities.

Huge water savings are achieved with the optional highly efficient water recycling system (HSG-WS model) that consumes only 5 liters of tap water per day

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Accessories and spare-parts for autoclave Tuttnauer 3870 HSG-WS-D

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A mains electricity (AC-powered) device designed to accept a biological sterilization indicator (bacterial spores) and to rapidly indicate whether adequate sterilization conditions have been achieved by testing the resistance of the biological indicator previously exposed to a sterilization process (e.g., by steam, or chemical sterilization). It is a bench-top unit intended to provide suitable incubation conditions for the biological indicator and to detect fluorescent enzymatic signals that correlate with the resistance of the biological indicator.

The Auto-reader is designed to incubate and automatically read Ultra Rapid Readout Biological Indicators, BT224, at 60°C for a final fluorescent result at 20 minutes. Biological indicators (BIs) from other brands are not compatible with this device and cannot be used.

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A sterilization indicator consisting of a standardized, viable population of microorganisms (e.g., bacterial spores known to be resistant to the mode of sterilization being monitored) that will display a change when exposed to an effective sterilization outcome indicating that sterilization parameters have been met. This is a single-use device.

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## ETMA Training mannequins - Mannequins pour formation

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The Emergency Care Learning Lab is designed to support training in basic emergency care for adults and children, including WHO's Basic Emergency Care program, the Community First Aid Responder program, and the Basic Critical Care program.

The kit is also ideal for CPR training, as it includes adult and infant CPR manikins, along with a free app that provides real-time feedback, interactive challenges, and tips to enhance CPR performance.

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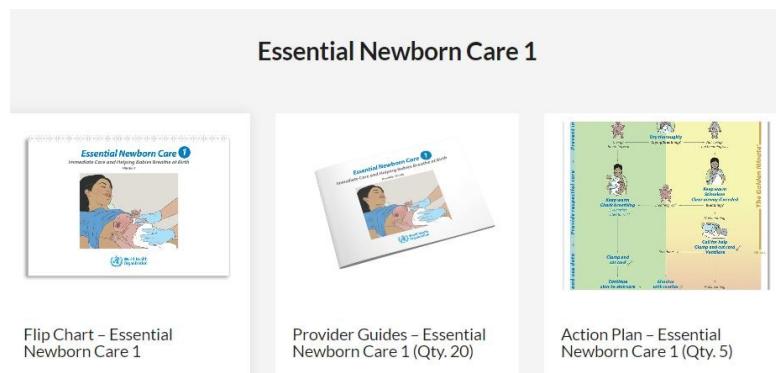
## ETPO

### Training posters – Planches murales

#### NEW / NOUVEAU

ETPOECSB2E-P	<b>STD</b>	ESSENTIAL NEWBORN CARE 1 Flip Chart English ESSENTIAL NEWBORN CARE 1 Flip Chart English
ETPOECSB3E-P	<b>STD</b>	ESSENTIAL NEWBORN CARE 1 Provider Guide English ESSENTIAL NEWBORN CARE 1 Provider Guide English
ETPOECSB4E-P	<b>STD</b>	ESSENTIAL NEWBORN CARE 1 Action Plan Poster English ESSENTIAL NEWBORN CARE 1 Action Plan Poster English

Essential Newborn Care 1 (ENC1) is a hands-on essential newborn care training program by the World Health Organization and the American Academy of Pediatrics. ENC1 is the first part of the WHO Essential Newborn Care Course (ENCC).

It focuses on the care of babies from birth to 60 minutes after birth. ENC 1 incorporates all content of the Helping Babies Breathe (HBB) program, updated to align with WHO recommendations and guidelines.

The ENCC is made up of three flexible components that allow organizers and facilitators to select the content that participants need and adapt the timeline for educational activities in the health facility.

- Flip chart ETPOECSB2E-P
- Provider Guides ETPOECSB3E-P
- Action plan ETPOECSB4E-P

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This new product is recommended for nutritional support among Pregnant and Breastfeeding women, providing the required balanced energy protein, and micronutrient requirements. The lipid-based nutrient supplement is a ready-to-use product intended for the prevention of Moderate Acute Malnutrition (MAM) in pregnant and breastfeeding women and girls. It also improves the healthy development of the foetus and the infant.

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Very specific and expensive product, intended for use only in specific cases that should be validated by nutrition and/or paediatric advisors.

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To simplify use in the field and minimize waste, the ENTERAL RTU hypercaloric normo-protein for child can be replaced by the ENTERAL RTU, isocal. child, normoprot, w/o fibre, 0.5l (NFOSENICNPW05) in most situations, in accordance with the new paediatric guidelines.

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This test had been removed because it is not manufactured anymore by Bio-Rad. Another NST option is available:  
SSDTLEIS24T2 LEISHMANIASIS Kala Azar TEST (GAD IT LEISH) ser/wb,1t 710124

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These 2 tests were made NST because not manufactured anymore. One NST option is available: SSDTBRUC50T TEST BRUCELLOSIS, Rose Bengal, ser, 1t (Spinreact 1200901).

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## STSS

### Transport, storage and sampling of biological material

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2 new swabs, created specifically for collection and transportation of suspected MPOX samples.

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New volume of an already existing STD article as the 250 g vials are not available anymore.

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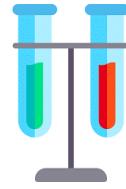
In order to avoid confusion between saline solution sterile used for eyes and non-sterile solution used in laboratory procedures, both articles were made NST and outdated and replaced by a STD item: SSCOSODC9A5 SODIUM chloride, 0.9%, sterile, 5ml, unidose, amp.

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## SBQC

## Bacterial strains Quality Control

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### CHANGE / CHANGEMENT

Several references for bacterial strains for quality controls had been completed with, in addition to ATCC strains, the possibility to NCTC or NCPF strains. The NCTC/NCPF strains are less expensive and has better availability than the ATCC strains.

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The manufacturer Sarstedt stopped the manufacture of the tube stripper used in the blood bags collection for transfusion. There are 2 NST alternative available in surgical family:

ESURTUMRB1 TUBE MILKING ROLLER, Brompton, autoclav.  
ESURTUMRGH1 TUBE MILKING CLAMP, Gunnar-Hey, 18 cm, autoclav.

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BLOOD BANK REFRIGERATOR (MB3000G), 230V, 100 bags 450ml

REFRIGERATEUR BANQUE DE SANG (MB3000G), 230V, 100poches 450ml

Because of environmental regulation, the STD blood bank MB3000G is not available anymore. Some NST options are available, check with your laboratory and biomedical advisors in order to choose the best option.

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New improved versions HIV GeneXpert cartridges are replacing previous cartridges, phased out in December 2024.

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**REMOVAL of tests and controls/RETRAIT de tests et contrôles**

ELAEHAET301	<b>NST</b>	(HemoCue Hb 301) CONTROL SOLUTION, kit 3 x 2 bottles (HemoCue Hb 301) SOLUTION DE CONTROLE, kit 3 x 2 flacons	Thermosensitive: 0208 IVD non-A/non-B
ELAEHAET302	<b>NST</b>	(HemoCue Hb 301) CONTROL SOLUTION, low, 2 x 1 ml vials (HemoCue Hb 301) SOLUTION DE CONTROLE, bas, 2 x 1 ml fl.	Thermosensitive: 0208 IVD non-A/non-B
ELAEHAET303	<b>NST</b>	(HemoCue Hb 301) CONTROL SOLUTION, normal, 2 x 1 ml vials (HemoCue Hb 301) SOLUTION DE CONTROLE, normal, 2 x 1 ml fl.	Thermosensitive: 0208 IVD non-A/non-B
ELAEHAET304	<b>NST</b>	(HemoCue Hb 301) CONTROL SOLUTION, high, 2 x 1 ml vials (HemoCue Hb 301) SOLUTION DE CONTROLE, haut, 2 x 1 ml fl.	Thermosensitive: 0208 IVD non-A/non-B

The kit assembled at ESC level and articles as pack of 2 vials are replaced by the articles identified as unit, already existing as STD:

- ELAEHAET306 (HemoCue Hb 301) CONTROL SOLUTION, low, 1ml vial
- ELAEHAET307 (HemoCue Hb 301) CONTROL SOLUTION, normal, 1ml vial
- ELAEHAET308 (HemoCue Hb 301) CONTROL SOLUTION, high, 1ml vial

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- ELAEHAET306 (HemoCue Hb 301) SOLUTION DE CONTROLE, bas, 1ml fl.
- ELAEHAET307 (HemoCue Hb 301) CONTROL SOLUTION, normal, 1ml fl.
- ELAEHAET308 (HemoCue Hb 301) CONTROL SOLUTION, haut, 1ml fl.

## NEW SPARE-PARTS/NOUVELLES PIECES DETACHEES

New spare-parts created for the centrifuge Hettich EBA200

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New spare parts created for the clinical chemistry analyzer Spotchem EZ

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---

Citric acid solution for water bath had been replaced by locally available cleaning vinegar 5 to 12%. Several NSL codes are existing in PHYG family:

- PHYGDETEV1- CLEANING VINEGAR white, 1l, 8-12%, bottle
- PHYGDETEV2- CLEANING VINEGAR, 250ml, 8-12%, bottle
- PHYGDETEV5- CLEANING VINEGAR white, 500ml, 8-12%, bottle
- PHYGDETEV5J CLEANING VINEGAR white, 5l, 8-12%, jerrycan

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---

ELAEBATC002	NST	(water bath) DETERGENT, bot. (bain-marie) DETERGENT, fl.	UN Code: to check Thermosensitive: CT30
-------------	-----	---	--

---

The detergent for water bath is replaced by denatured alcohol or isopropyl alcohol 70% for surface; locally available

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More thoracic drains are added in this module on request of the surgical working group as the qty was not enough for the initial phase of an active conflict.

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Change of the number of disposable drape sets: more universal sets (ELINDRSS5SS) are needed but fewer orthopaedic sets (ELINDRSS3OR). The Universal sets can be used also for orthopaedic procedures and are easier to install in terms of IPC.

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Change of content: guedels become sterile (same qty)

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Change of content: IV Cath 24G different technical specifications (removal of the injection port), same qty

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Change of content: the red identification bracelets (SMSUBRAI1R-) are replaced by the white ones for inpatient (SMSUBRAI3W-), same qty

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KMEDKHCX3OP	<b>STD</b>	OPD, COMPLEMENTARY PART, observation beds, complete OPD, PARTIE COMPLEMENTAIRE, lits observation, complet
KMEDMHAE221	<b>STD</b>	(mod AMP) COMPLEMENTARY RESUSCITATION EQUIPMENT (mod PMA) MATERIEL VENTILATION COMPLEMENTAIRE
KMEDMHCE23-	<b>STD</b>	(mod OPD) MEDICAL EQUIPMENT 2021 (mod consultations) EQUIPEMENT MEDICAL 2021
KMEDMHDE31-	<b>STD</b>	(mod delivery & neonate) MEDICAL EQUIPMENT 2021 (mod accouchement & néonatal) EQUIPEMENT MEDICAL 2021
KMEDMHEE321A	<b>STD</b>	(mod emergency) COMPLEMENTARY RESUSCITATION EQUIPMENT 2021 (mod emergency) COMPLEMENTARY RESUSCITATION EQUIPMENT 2021
KMEDMHIE21-	<b>STD</b>	(mod ICU) EXAMINATION-RESUSCITATION EQUIPMENT (mod USI) EQUIPEMENT D'EXAMEN-REANIMATION
KMEDMHWE221	<b>STD</b>	(mod ward) COMPLEMENTARY RESUSCITATION EQUIPMENT (mod hospit) MATERIEL VENTILATION COMPLEMENTAIRE
KMEDMNUTI35	<b>STD</b>	(module nut. inpatient) RESUSCITATION ITEMS 2021 (module nut. hospitalisation) ARTICLES REANIMATION 2021

Change of content: the PULSE OXIMETER (Masimo RAD-5) + accessories (EEMDPOXE4--) is replaced by the PULSE CO-OXIMETER (Masimo RAD-57) + accessories (EEMDCAPE2--), same quantity.

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The transport of CHLORHEXIDINE digluconate 4%, soap, 500ml, bot. (DEXTCHLHSP4) and PERMETHRIN, 5%, cream, tube (DEXTPERM5T-) is regulated in some countries. These two products have been removed from the module KMEDMNUTI36A and included in the module KMEDMNUTI36D.

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Creation of the new module (module nut. outpatient) MEDICINES, RTR 2021 (KMEDMNUTO35D-) including CHLORHEXIDINE digluconate 4%, soap, 500ml, bot. (DEXTCHLHSP4), for which transport is regulated in some countries.

The CHLORHEXIDINE has been removed from the module KMEDMNUTI35A.

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Both kits MOBILE MEDICAL BAG KIT, polypropylene bag (KMEDKFAI5--) and MOBILE MEDICAL BAG KIT, rucksack (KMEDKFAI5RS) have the same content. The rucksack model is much more ordered than the propylene bag.

Critical Care Referents have requested to keep with only one model: the rucksack (KMEDKFAI5RS) and make the polypropylene bag NST outdated (KMEDKFAI5--).

Replacements have been made in the kits concerned.

The drugs and basic medical equipment are intended for use in exploratory missions and can serve as a first aid kit in vehicles. This kit is not meant to be the team's personal pharmacy. Only medical or paramedical staff are allowed to use its contents.

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