## INSTRUCTIONS FOR USE

|             |  |
|-------------|--|
| KWATKBOD25- | <b>KIT, DEAD BODY MANAGEMENT, 25 bodies</b><br><b>KIT PRISE EN CHARGE CADAVRES, 25 corps</b> |
|-------------|--|

Use gloves, mask and goggles throughout this process. Operatives should also be wearing boots.

### **Use**

- Fill the buckets with clean water
    - o Add 2 spoons **per 1 liter** of clean water (= 36 g of NaDCC granules or about 30 g of HTH granules) to the red bucket this gives a 2 % chlorine solution for disinfecting the dead bodies and soaking the cotton
    - o Add 1 spoon of NaDCC granules (or HTH) in each white bucket **with 20 liters** of clean water, this gives a 0.05 % solution for washing hands and materials after the process is over. **THIS SHOULD NOT BE USED FOR DRINKING!**
  - Fill the measuring jug with the 2 % solution from the red bucket
  - Pour the solution on the dead body and on any vomit or faeces around the body.
  - Tear off pieces of cotton in 10 cm x 10 cm squares. Soak in 2% solution and insert in the dead body's mouth, nose and anus
  - Pour some solution in the inside of the body bag and then place the body inside the bag. Close the zip
  - Pour some disinfectant solution with the measuring jug on the surface where the body was resting before being placed in the body bag
  - Pour also some solution on the outside of the body bag so it is ready for transportation. If the body is to be transported ensure that the zip remains upright at all times to avoid leakages via the zip which is not watertight
- Dry a part of the body bag with toilet paper and mark the name of the patient using the marker.

### **Maintenance**

Disinfect boots and gloved hands and apron with the red bucket 2 % chlorine solution. Avoid direct contact between the chlorine solution and the skin or clothes as it may cause burns

Take off mask and gloves and dispose off (for burning or incineration)

Other reusable equipment (buckets, jug...) must dry before storing it.

All persons involved with the procedure should wash their hands with the white bucket 0,05 % chlorine solution

### **Precautions for use**

Never mix the NaDCC solution with HTH powder: this can provoke a chemical reaction that could be explosive!

In some context cultural habits have to be considered and an anthropological support might be required

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|                    |  |
|--------------------|--|
| <b>KWATKBOD25-</b> | <b>KIT, DEAD BODY MANAGEMENT, 25 bodies</b><br><b>KIT PRISE EN CHARGE CADAVRES, 25 corps</b> |
|--------------------|--|

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