

# UTILISATION DU MODULE PRELEVEMENT CHOLERA (KMEDMSAM1C-)

1. Ouvrir le tube contenant déjà une pastille
2. Prendre la pastille avec la pince propre  
***Désinfecter et laver la pince entre 2 prélèvements (flamber la pince et laisser refroidir ou désinfecter avec du chlore et bien rincer avec de l'eau filtrée) Alternative =pour éviter de laver la pince entre chaque prélèvement certains utilise des aiguilles à usage unique pour prendre la pastille de papier filtre.***
3. Tremper la pastille dans la selle suspecte (ou vomissement).  
***Important: vérifier que la selle n'a pas été chlorée auparavant.***
4. Remettre la pastille dans le tube.
5. Rajouter 3 gouttes de sérum physiologique.
6. Refermer le tube correctement.
7. Identifier correctement le tube en y mettant le lieu et la date du prélèvement.
8. Joindre les informations cliniques et épidémiologiques sur une feuille de renseignement en indiquant l'identité du patient (Nom, âge, sexe...), traitement reçu...
9. Faire parvenir le tout à température ambiante par des moyens sûrs au service médical

Hermétiquement fermé, le prélèvement peut se conserver 2 semaines à température ambiante.

## INSTRUCTIONS FOR USE OF THE CHOLERA SAMPLING KIT (KMEDMSAM1C-)

1. Open the tube, in which a disc has been placed.
2. Remove the disc with a clean pair of tweezers.  
***Disinfect and wash the tweezers between specimens (sterilise the tweezers over a flame and let it cool down or disinfect with chlorine and rinse thoroughly with filtered water)***  
***Alternative to the tweezers: use single use needles***
3. Dip the disc into the faeces or vomit sample.  
***Important: make sure that the faeces has not been chlorinated beforehand.***
4. Place the disc back in the tube.
5. Add three drops of physiological salt solution.
6. Seal the tube.
7. Label the tube - place and date of collection.
8. Do not forget to include the clinical and epidemiological documentation with the patient's details (name, age, sex, treatment, etc.)
9. Send to the medical department (at room temperature).

If the specimen has been hermetically sealed, it can be stored for 2 weeks at room temperature.